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## User Manual

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**Please read this user manual carefully , before you start using the Safety Clamp for PleurX® (hereinafter referred to as „Safety Clamp“).**

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Telefon 07471-739 72-0

Do you have a question or need any help?

We will be pleased to answer your questions.

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**en**

### **Table of Contents**

Safety Clamp for PleurX®	7
General data	7
Area of application	8
Description	8
Safety Instructions	9
Instructions for use	9
Cleaning instructions	10
Overview drainage products	11

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## Safety Clamp for PleurX®

The Safety Clamp protects the connection between a PleurX® catheter valve and the PleurX® and ewimed drainage kits and prevents an accidental disconnection. The product is reusable (no disposable).

### The Safety Clamp offers following advantages:

- Unintentional disconnection of the PleurX® catheter valve and a PleurX® drainage kit is prevented.
- Protects the connection from contamination.
- No need to further secure the connection with e.g. adhesive tape.

### General data

**REF** P1100      Safety Clamp for PleurX®, 1 PU = 10 pieces, unsterile

**REF** P1100S      Safety Clamp for PleurX®, 1 PU = 2 pieces, unsterile

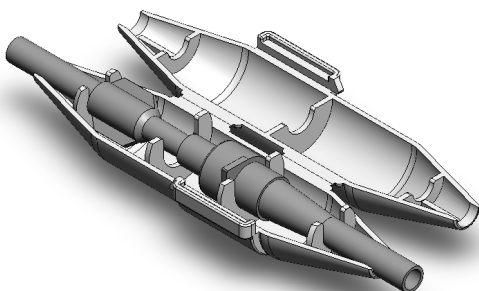
Fig. 1:  
Safety Clamp  
open with inlying connection of  
a PleurX® catheter valve and  
drainage kit

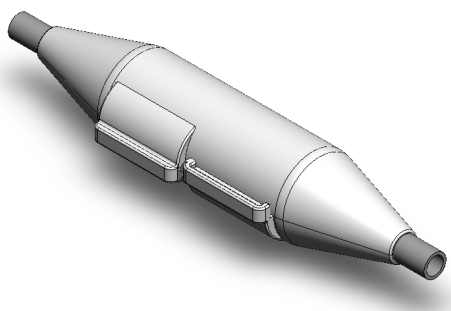
Fig. 2:  
Safety Clamp closed with inlying  
connection of a PleurX®  
catheter valve and drainage kit

- The product is delivered unsterile
- The product is reusable (no disposable)
- possibility of reprocessing (See also p. 10)
- Material:            HDPE
- Minimum purchase: 1 PU

## Area of application

To secure the connection between a PleurX<sup>®</sup> catheter valve and PleurX<sup>®</sup> or ewimed drainage kit:

- PleurX<sup>®</sup> drainage kit
- PleurX<sup>®</sup> Vacuum Bottle
- ewimed drainage kit
- ewimed reservoir

## Description

The safety clamp secures the connection between a PleurX<sup>®</sup> catheter valve and PleurX<sup>®</sup> and ewimed drainage kits. To fix the connection put it in the part of the clamp with the four ridges. To prevent a disconnection or contamination of the connection close the Safety Clamp by pushing the two parts together.

Basically the connection between the catheter valve and the PleurX<sup>®</sup> and ewimed drainage kits is secured with a notch (click). However, some customers prefer a mechanical fixture to prevent an accidental disconnection. Until now, this was done by attaching adhesive tape around the valve. With this Safty Clamp it is not necessary to further secure the connection with adhesive tape.

## Safety Instructions

1. Please make sure that the tubing connection lies properly in the slip to prevent the tubing from squeezing. If you squeeze the tubing connection and accidentally damage it, please use the blue emergency slide clamp in the procedure pack pouch contained in the PleurX® and ewimed drainage kit to close the catheter (see user manual PleurX® and ewimed drainage kit).
2. The safety clamp is delivered and used unsterile, since it has no contact with any fluid carrying surfaces. It is possible to clean the safety clamp, but it does not have full thermal durability.  
Please follow the cleaning guidelines on the following page.
3. Please observe the hygienic instructions of your clinic.
4. When using the safety clamp in order to drain pleural or ascites fluid through a PleurX® or ewimed drainage kit, please observe the corresponding user manual.
5. Do not use this safety clamp if it is damaged (e.g. defected pawls / the clamp does not close properly, parts of the clamp have broken off, surface corrosive action or unaesthetic optic due to aging).

## Instructions for use

1. Put the catheter with the connected drainage line in the safety clamp so that the connection lies in the part of the clamp with four ridges. It does not matter in which direction the connection lies.
2. Close the clamp by pushing the two parts together.
3. Open the pawls after use and pull the two parts apart.

## Cleaning instructions

### Treatment options:

1. Disinfection by cleaning / disinfection device: Using detergents, activating agent and desinfectant according the usual procedures (90 degrees). Please observe the manufacturer's information.
2. Disinfection by disinfecting solution: Depending on the concentration and type of disinfecting solution, please observe the manufacturer's information concerning the exposure time.
3. Spray disinfection: Treatment by disinfectant spray for surfaces is possible but sterility is not given and therefore not recommended.

Due to international standards and national guidelines just validated mechanical treatments for cleaning and disinfection should be used.

### **Note:**

Autoclaving is not recommended. Discoloration of the material or damages because of the high temperatures are possible.  
Precleaning in an ultrasonic bath is possible.

Do not use this safety clamp if it is damaged (e.g. defected pawls / the clamp does not close properly, parts of the clamp have broken off, surface corrosive action or unaesthetic optic due to aging).

In perfectly clean condition and if the Safty Clamp works properly it can be used permanently.

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Hersteller / manufacturer

ewimed  
Dorfstraße 24  
72379 Hechingen

PleurX® is a trademark and/or registered trademark  
of CareFusion Corporation, or one of its subsidiaries.

ewimed  
Dorfstraße 24  
D - 72379 Hechingen  
t 07471-739 72-0  
f 07471-739 72-13  
info@ewimed.com  
www.ewimed.com

**ewimed**