ja

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Do not use if package is damaged.

© 2010, 2013, 2014, CareFusion Corporation or one of its subsidiaries. All rights reserved.

Sponsored by
CareFusion Australia 316 Pty. Ltd.
CareFusion New Zealand 313 Ltd.
361-30505 • 2014-04

Proofed by: _____ Date: _____
Dimensions checked: _____ Copy checked: _____

RC0414206
Vernon Hills, IL
Richard Cisneroz
05-20-14
361-30505

en

Instructions for Use

Caution: The introducer needle must be withdrawn as soon as fluid is observed in the 5 ml syringe to avoid potential damage to internal organs. Care must also be taken to avoid damaging internal organs with the catheter tip. The introducer needle should **not** be advanced in the catheter after it has been partially withdrawn.

Important: The self-sealing valve is designed to prohibit the introducer needle from being reinserted into the device after it has been fully withdrawn.

A. Introducer Needle **D.** 5 ml Syringe
B. Catheter **E.** Drainage Tube
C. Self-Sealing Valve **F.** Vacuum Needle 1-1/2" (3.8 cm)
G. #11 Scalpel

Procedure

After puncture site has been prepared and anesthetized:

1. A scalpel can be used to make a small incision.
2. Hold the stopcock and remove the needle protector. Insure that the sharp needle point extends beyond the catheter tip.
3. Puncture the skin and introduce the catheter-needle assembly for either: **Thoracentesis** into the pleural space or **Paracentesis** into the peritoneal cavity.
4. Advance the catheter-needle assembly until fluid can be aspirated through the needle into the 5 ml syringe.
5. Withdraw the introducer needle completely and insure the stopcock handle is off to the side port.
6. Position catheter as required.

These steps apply when a syringe is used for fluid collection:

- A. Attach a syringe to stopcock side port using a push-twist motion.
- B. Turn the stopcock off to the self-sealing valve.
- C. Collect fluid.

These steps apply when a vacuum bottle is used for fluid collection:




- A. Attach the drainage tubing to stopcock side port using a push-twist motion.
 - B. Remove the protector from the vacuum needle and puncture the bottle.
 - C. Turn stopcock off to self-sealing valve. This will start fluid flow into the bottle.
8. Upon completion of procedure, turn the stopcock off to the side port.

9. Remove the catheter from the patient and protect the puncture site with a bandage.

These instructions are supplied to explain the use of components in this kit. Procedural details will vary according to the physician's clinical judgment.

Caution: For single use only. Re-use may result in a non-functional product or contribute to cross contamination.

Note: Not made with natural rubber latex.

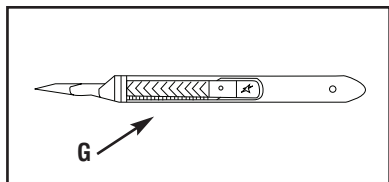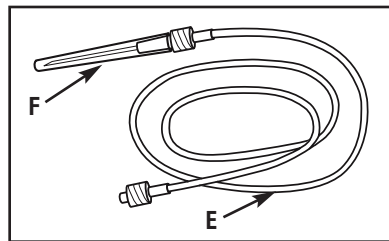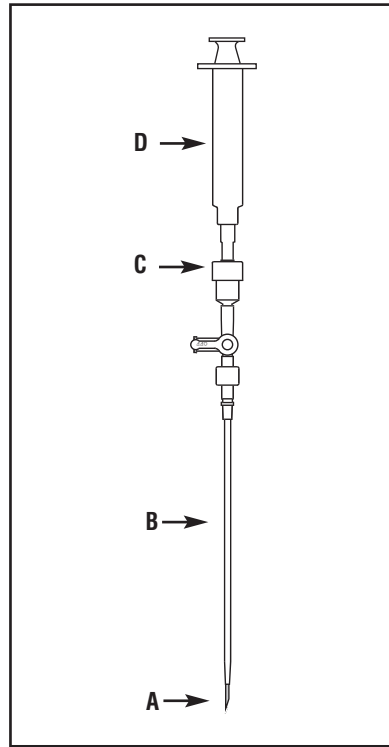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