de ewimed Drainage-Set, 2000 ml

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en ewimed Drainage Kit, 2000 ml

Instructions for use

fr ewimed kit de drainage, 2000 ml

Instructions

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istruzioni d'uso

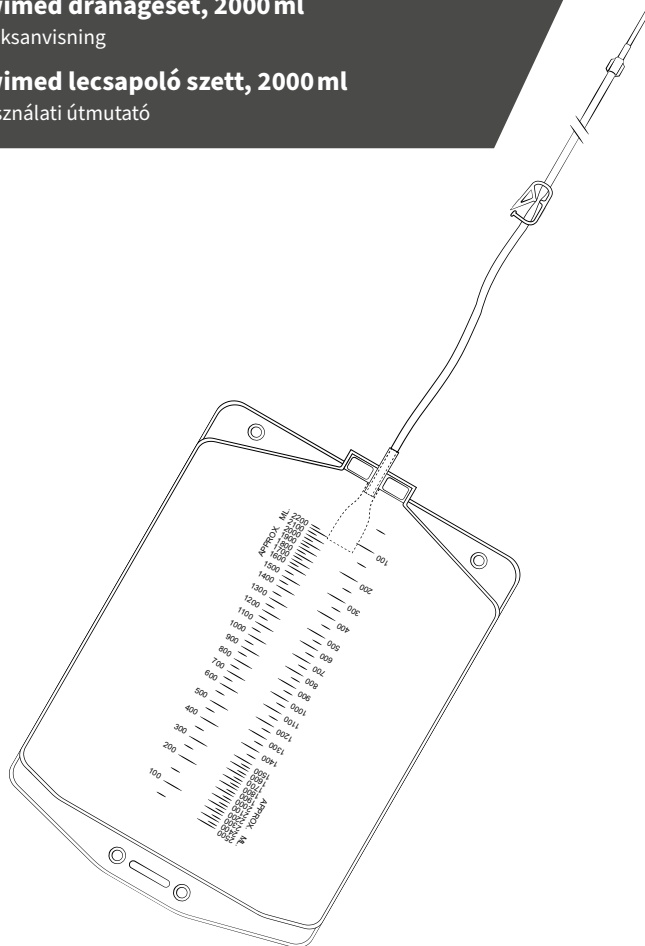
sv ewimed dränageset, 2000 ml

Bruksanvisning

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ewimed

REF

50-7505

de Erklärung der Symbole
en Explanation of Symbols
fr Explication des symboles

it Spiegazioni dei simboli
sv Förklaring av symbolerna
hu Szimbólumok magyarázata

de Bitte beachten Sie, dass nicht alle dargestellten Symbole auf Ihr Produkt zutreffen. Die jeweilig zutreffenden Symbole finden Sie auf dem Etikett Ihres Produktes. / **en** Please note that not all symbols shown apply to your product. The relevant symbols can be found on the product label. / **fr** Veuillez noter que tous les symboles décrits ne s'appliquent pas à votre produit. Vous trouverez les symboles correspondants sur l'étiquette de votre produit. / **it** Si prega di notare che non tutti i simboli mostrati sono applicabili al vostro prodotto. Troverete i simboli pertinenti sull'etichetta del vostro prodotto. / **sv** Observera att inte alla avbildade symboler är tillämpliga för din produkt. Vilka symboler som gäller för din produkt hittar du på produktetiketten. / **hu** Felhívjuk szíves figyelmét, a felsorolt szimbólumok közül nem mindegyik kapcsolódik az ön termékéhez. A termékére vonatkozó szimbólumokat annak címkéjén találja.

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Content

- 2 Explanation of Symbols
- 6 Drainage Log Chart
- 7 Product Description
- 25 The drainova® Catheter, drainova® ArgentiC Catheter, PleurX™ or PeritX™ Catheter
- 25 The ewimed Drainage Kit
- 26 Warnings
 - general
 - in combination with the catheter
- 28 Preparation
- 28 Drainage Procedure
- 31 Final steps
- 31 Applying a new dressing
- 33 Emptying the Reservoir
- 34 Frequently Asked Questions
- 35 Returns / Refunds
- 35 Glossary

You should have already been instructed in the correct use of the components in this set either by ewimed training staff, district nursing staff or a doctor. Please contact your doctor or a member of the nursing staff if you are experiencing difficulties or have questions relating to effusion drainage.

Please carefully read the instructions before using the ewimed Drainage Kit.

All explanations of terms (*) can be found in the glossary on page 35.

You have questions or need help?

Contact us:

DE: +49 7471 73972-0
info@ewimed.com

AT: +43 2231 22500
info@ewimed.com

EN: +49 7471 73972-0
info@ewimed.com

SE: +46 8 25 11 69
info.se@ewimed.com

CH: +41 52 577 02 55
info.ch@ewimed.com

HU: +36 1 998 8295
info@ewimed.hu

Further detailed information as well as drainage training videos can also be found on our homepage at www.ewimed.com.

The drainova® Catheter, drainova® ArgentiC Catheter, PleurX™ or PeritX™ Catheter

The drainova® Catheter, drainova® ArgentiC Catheter, PleurX™ or PeritX™ Catheter serve to drain fluid from the chest cavity (pleural effusion*) or the peritoneal cavity (ascites*).

Your doctor has implanted this catheter so that you can independently drain the pleural effusion* or the ascites* at home according to your needs - rendering repeated visits to hospital for puncturing unnecessary.

The implanted part of the catheter is perforated with drainage holes, which allow the effusion to drain out. During implantation, the polyester cuff is placed in a subcutaneous tunnel to prevent infections*. The safety valve on the visible part of the catheter is generally closed, preventing both leakage of effusion and air from entering.

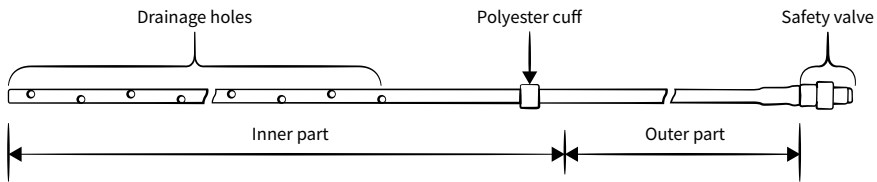
Illustration of a drainova® catheter

A protective cap or a drainova® silicone cap is attached to the safety valve to protect it from contamination or damage when not draining.

To drain effusion fluids use the drainova® Drainage Set or the ewimed Drainage Set. The key-lock-principle ensures the optimal connection of the drainage sets with the safety valve.

The drainova® Catheter, drainova® ArgentiC Catheter, PleurX™ Catheter or PeritX™ Catheter will hereinafter be referred to as 'catheter'.

The ewimed Drainage Kit

The ewimed Drainage Kit contains sterile materials that you will need for the safe and easy drainage of effusion with your catheter.

The gravity reservoir contained in the ewimed Drainage Kit will hereinafter be referred to as the 'reservoir'.

All products are CE labelled and have been sterilized with ethylene oxide gas (see individual packaging).

For the drainage procedure, we additionally recommend:

- Alcohol-based skin disinfectant spray
- Hand disinfectant
- Non-sterile disposable gloves

Warnings

general

Do not use this product if:

- The sterile packaging is damaged
- The product is damaged
- The expiry date has expired

The ewimed Drainage kit is only suitable for use for the intended purpose.

The consumables are intended for single use only!

The ewimed Drainage Kit is delivered in a sterile condition, which is ensured by appropriate quality controls. The Drainage Kit must not be re-sterilised.

The product is for single-use only. Re-use can lead to cross-contamination.

ewimed GmbH accepts no liability for products that have been re-used, re-sterilised or removed from damaged packaging.

Drainage from the chest cavity

Follow your doctor's instruction regarding the drainage amount.

ewimed recommends draining no more than 1000 ml of effusion per day. Drainage of larger amounts of effusion should only be carried out on your doctor's instruction. You may feel an unpleasant pulling sensation during drainage. Should this become too intense, close the pinch clamp on the connection tube to pause the flow.

Possible complications due to recurrent drainage of pleural effusion* include pneumothorax*, re-expansion pulmonary edema*, hypotension*, circulatory collapse*, septa* in the chest cavity or infection*.

Peritoneal drainage

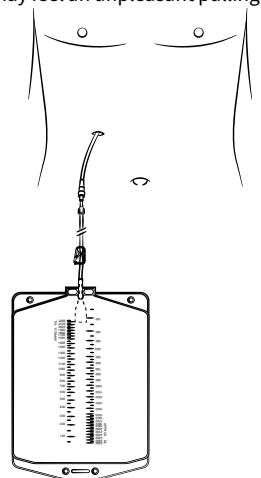
Follow your doctor's instruction regarding the drainage amount.

ewimed recommends draining no more than 2000 ml of effusion per day. Drainage of larger amounts of effusion should only be carried out on your doctor's instruction. You may feel an unpleasant pulling sensation during drainage. Should this become too intense, close the pinch clamp on the connection tube to pause the flow.

Possible complications due to recurrent drainage of ascites* include hypotension*, circulatory collapse*, electrolyte imbalance*, protein loss*, hepatorenal syndrome*, leakage of ascites* through the incisions*, peritonitis*, septa* in the peritoneal cavity* and infection*.

Position of the reservoir during drainage

As the effusion is drained by gravity, make sure that the reservoir is always situated below the catheter exit site during drainage.

Cleaning/Disinfection

If effusion is leaking out and comes in contact with skin, clean the skin with soap and water. For surfaces, use suitable cleaning products or surface disinfectant. Do not use Octenisept or disinfectants containing iodine for cleaning/disinfecting the catheter or safety valve.

The alcohol wipes are highly flammable.

Keep the wipes away from naked flames.

Warnings

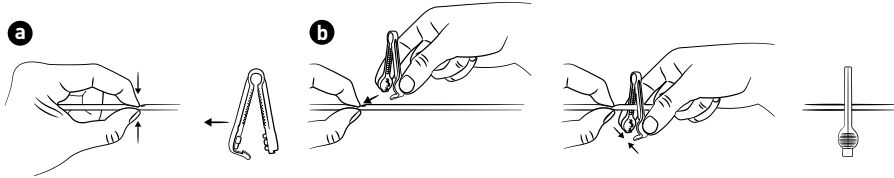
in combination with the catheter

Never cut through the catheter or cut the safety valve off

Avoid contact of sharp objects with the catheter.

If the function of the safety valve can no longer be guaranteed or if the safety valve has been accidentally cut off or is no longer present, proceed as follows:

- Firmly squeeze the catheter between your fingers.
- Hold the emergency clamp with the other hand and close the clamp with the catheter inserted until a click is heard.
- Immediately contact the clinic where the implantation took place.

Use only original accessories.

Only original accessories may be connected to the catheter. Do not insert anything other than the specially-designed access tip into the safety valve, otherwise the safety valve may become damaged. Improper handling can cause air to enter the body or fluid to permanently leak out.

Keep the safety valve on the catheter clean and the access tip on the connection tube sterile.

Ensure that they do not come into contact with non-sterile objects to avoid contamination. The safety valve must be disinfected before each drainage.

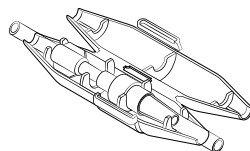
When draining, ensure that the access tip and the safety valve are securely connected. Make sure that the connection tube is not dragged or pulled.

Connect the access tip to the safety valve in a sterile manner and make sure that they are firmly connected. Accidental disconnection may result in contamination. If this happens, disinfect the safety

valve using a new alcohol wipe and discard the disconnected drainage material. A new sterile Drainage Kit must be used to continue drainage.

Please observe the local, regional and national waste disposal regulations.

Please note: If required (for additional protection against accidental disconnection) the connection can be secured with the separately available drainova® clickFix (Art. No. P1100) .

The drainova® clickFix can be additionally prescribed. It is not contained in the Drainage Kit.

Preparation

The effusion should be drained with an ewimed Drainage Kit according to your doctor's instructions. If you have not received a medical recommendation for the drainage interval or drainage volume, you will receive a recommendation during training.

1. Prepare a clean, clear work area on a table or other work surface.

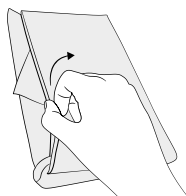
Please note: When disposing of used material, please observe the local, state and national regulations . If necessary, place a waste container ready.

2. Disinfect your hands. We also recommend washing your hands thoroughly beforehand.
3. Put on the non-sterile disposable gloves.
4. Remove the dressing covering the catheter. Carefully lift off one corner of the dressing and slowly pull away from the skin. Make sure not to accidentally pull on the catheter.
5. Take off the gloves and disinfect your hands again.
6. Open the sterile packaging of the ewimed Drainage Kit.
7. Open the sterile packaging of the dressing set by pulling the two sides of the bag apart at the top.


Drainage Procedure

1. Place the dressing set on the work surface with the flap facing upwards. Carefully unfold the blue drape. Leave the items contained on the drape.

The items in the dressing set are sterile and must not come into contact with non-sterile objects.

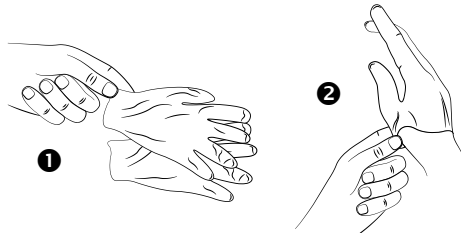2. Holding the gloves at the opening, put them on as shown. Both gloves are suitable for both hands. Make sure that no non-sterile objects, such as skin or clothing, are touched with the gloves.

en

- 2.1 Tear open the alcohol wipe sachets on one side. Do not remove the alcohol wipes from the sachets. Open each sachet in preparation to place them onto the safety valve later on and place the sachets back on the edge of the blue drape.
3. Loosen the drainova® silicone cap or the PleurX™/PeritX™ protective cap:

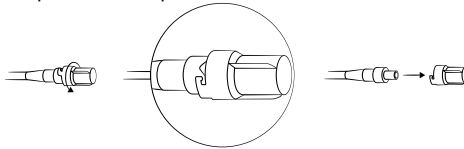
- **When using the drainova® silicone cap:**

Firmly hold the safety valve of the catheter and carefully remove the drainova® silicone cap. Dispose of the cap.


- **When using the PleurX™/PeritX™ protective cap:**

Firmly hold the safety valve of the catheter, turn the protective cap anticlockwise and carefully remove. Dispose of the cap.

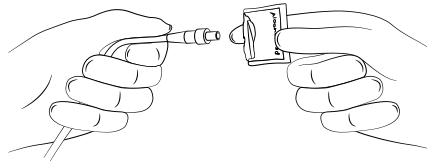
4. Insert the safety valve into one of the already opened alcohol wipe sachets. Gently rotate to clean the safety valve. Leave the safety valve in the sachet while carrying out steps 5 + 6.

Important: Do not use Octenisept® or disinfectants containing iodine.

5. Open the packaging and take the reservoir out.

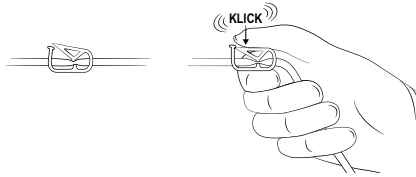
The reservoir in the packaging is sterile. It is important that the access tip on the connection tube remains sterile. Place the reservoir next to the blue drape and hold the access tip at the base in one hand. Completely close the pinch clamp on the connection tube with your other hand.


6. Whilst continuing to hold the access tip at the base, carefully pull the protective cover off the access tip and discard.

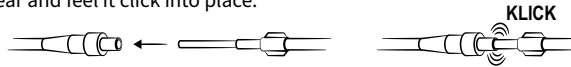- 6.1 Remove the alcohol wipe sachet from the safety valve and insert the access tip into the safety valve. Make sure that the safety valve and the access tip are firmly connected. You can hear and feel it click into place.

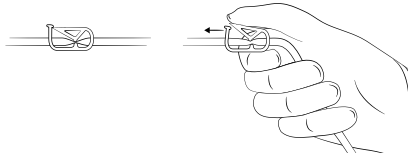7. The reservoir should always be kept underneath the catheter exit site, as drainage is aided by gravity.
8. Open the pinch clamp on the connection tube so that drainage can begin.


Please note: Flow can be slowed or interrupted if:

- The effusion is almost completely drained
- The effusion has a high viscosity
- The safety valve is blocked by a cogel* or protein thread


Additional information on how to proceed in such a case can be found in the 'Frequently Asked Questions' section starting on page 34.

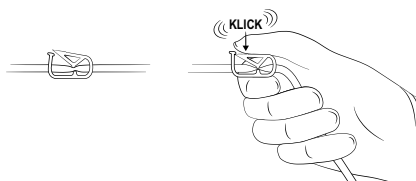9. When the drainage volume recommended by the doctor has been reached, stop drainage by closing the pinch clamp. Make note of the drainage volume in the 'Drainage Log Chart' on page 6.


Final steps

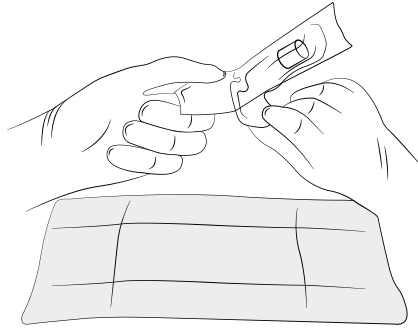1. Open the drainova® silicone cap packaging and let the cap drop onto the blue drape.

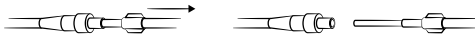
2. Take the connection tube in one hand and the safety valve in the other hand. Pull the access tip out of the safety valve in one firm, smooth motion. Set down the connection tube.

3. Clean the safety valve with an alcohol wipe for about 30 seconds and then dispose of the wipe.


Important: Do not use Octenisept® or disinfectants containing iodine.

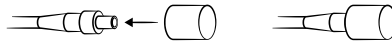4. Attach the new sterile drainova® silicone cap firmly onto the safety valve of the catheter. The cap will remain in place without the need for additional fixation.


Applying a new dressing

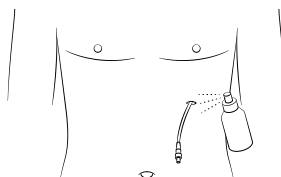1. Clean the area around the catheter exit site with the alcohol-based skin disinfection spray. Ensure an exposure time of 30 seconds. Please also ensure that afterwards the catheter exit site and the skin around the catheter are dry. Only then apply the new dressing.

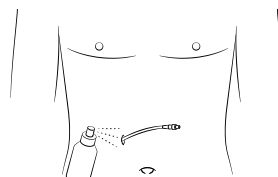
Disinfecting the pleural catheter exit site


Disinfecting the ascites* catheter exit site

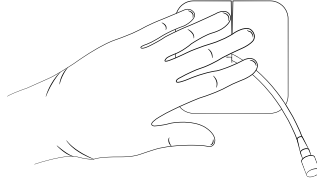
2. Place the foam pad around the catheter with the slit facing upwards.

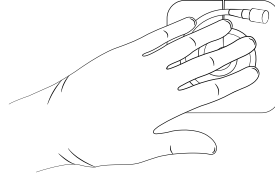
3. Loop the catheter and position it on the foam pad.

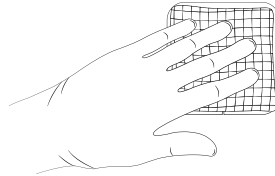
4. Cover the catheter with the gauze pads.

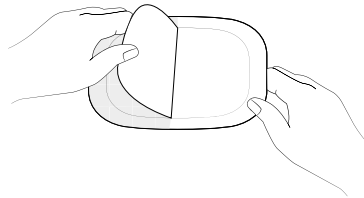
5. Take off both gloves.
6. The self-adhesive film dressing is comprised of three layers.
 - I. Printed protective film (two-part)
 - II. Self-adhesive film dressing (transparent)
 - III. Unprinted protective film (three-part)
7. Detach the middle part of the printed protective film from the film dressing.

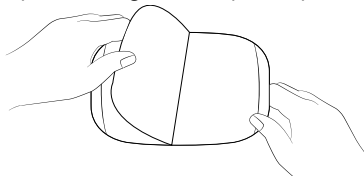8. Turn the film dressing over and expose the self-adhesive surface by peeling off the unprinted protective film. Leave both strips on the edges of the unprinted protective film stuck.

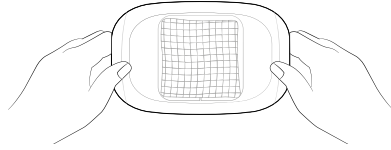
9. You can hold the film dressing securely by the two edge strips, position it over the gauze compresses and stick it to the surrounding skin.

10. Carefully remove both edge strips of the unprinted protective film and the frame of the printed protective film. Smooth out the entire self-adhesive film dressing from the center to the edges.

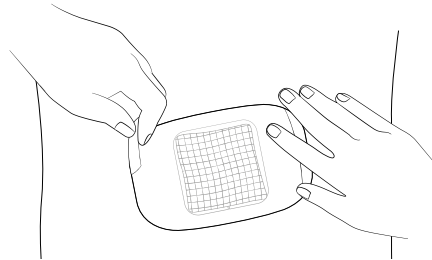
Please note: Dispose of used dressing material in accordance with local regulations.

en

Emptying the Reservoir

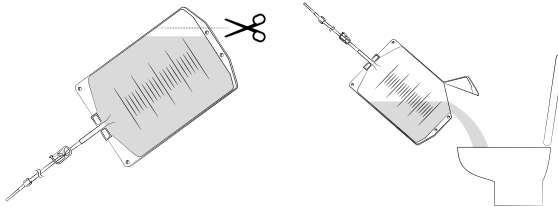1. Cut off a corner of the reservoir using scissors and empty the contents into the toilet.

Please note: Used products and effusion fluid may pose biological risks. Therefore, please observe local and national regulations when disposing of used products and effusion.

2. Place the reservoir in a plastic bag, seal it firmly and dispose of the bag in accordance with local regulations.

Frequently Asked Questions

When can the sutures (stiches) be removed?

The incision sutures can typically be removed after 7–10 days. The suture holding the catheter in place however, should not be removed before 30 days have elapsed.

How frequently should I drain?

Follow your doctor's instructions regarding the drainage quantity.

The effusion fluid should be drained using a Drainage Kit every one or two days or as instructed by the doctor. Consult your doctor if you want to drain more or less frequently.

ewimed recommends draining a maximum of 1000 ml of effusion from the chest cavity.

ewimed recommends draining a maximum of 2000 ml of effusion from the abdominal cavity.

What should I do if I still feel short of breath after draining?

If you still have difficulty breathing after successfully draining effusion fluid from the chest or abdominal cavity, inform your doctor.

What should be done if the effusion volume decreases or no effusion can be drained at all?

There can be two reasons for this:

1. If the amount of fluid during the last drainage deviates greatly from the usual amount or if no effusion at all can be drained, the catheter may be blocked or a protein thread may be obstructing the access tip, preventing drainage. Try to remove the blockage by squeezing and using stroking movements along the catheter.

If you are unable to remove the blockage, consult your doctor or contact the company central office (see contact details on page 24).

2. If the drainage amounts continuously decrease and you have no complaints or weight gain, it is possible that no effusion has formed. Discuss next steps with your doctor.

Can I shower or take a bath with the catheter?

If the catheter is completely and correctly covered with the self-adhesive dressing, you can shower. If however, the gauze pads get wet when showering, remove the dressing, clean and dry the area and apply a new dressing. The catheter should not be immersed under water in a bathtub or swimming pool. Avoid direct contact of the safety valve with bath or shower water.

Please note: We recommend showering before drainage / changing the dressing, so that the dressing can be promptly changed if it gets wet.

What should I do if I accidentally pull out the catheter?

The catheter should be adhered to the subcutaneous* skin tissue by the polyester cuff. If the catheter is accidentally pulled out and the cuff is visible, cover the catheter exit site with a sterile dressing and immediately contact the clinic where the implantation took place.

How can I tell if I have an infection*?

Immediately inform a doctor if you experience pain, skin redness (erythema*), palpable heat, swelling (edema*), fever or fluid retention as these could be signs of infection*.

Proceed in the same way should you notice a change in colour of the drained effusion.

Redness and slight pain are to be expected shortly after implantation, but these should not persist or get worse.

Returns/Refunds

Only originally packed (unopened, unmarked or damaged) cartons containing Drainage Kits will be accepted for return and refund.

Glossary

Ascites	Effusion or accumulation of fluid in the abdomen (stomach)
Circulatory collapse	Temporary loss of consciousness
Clot	Blood clot
DEHP	Plasticizers in plastics
Edema	Swelling due to excessive fluid accumulation
Electrolyte imbalance	Disturbance of the balance of ions (e.g. sodium, potassium, calcium) in the blood or cells in the body
Erythema	Skin redness
Heimlich valve	Backflow prevention for increased patient safety
Hepatorenal syndrome	Decrease in renal function
Hypotension	Low blood pressure
Incision	Skin incision during the operation e.g. for the implantation of the catheter
Infection	Reaction of the body's own defence system to pathogens such as bacteria, viruses etc.
Peritoneal cavity	Abdominal cavity

Peritonitis	Infection of the peritoneum
Phthalate	Plasticizer (e.g. DEHP)
Pleural effusion	Effusion or fluid accumulation in the pleural cavity
Pleura	Thin, serous skin in the thoracic cavity (pleura)
Protein loss	Reduction of the amount of protein in the blood (protein depletion)
Reexpansion pulmonary edema	Accumulation of fluid in functional lung tissue due to the sudden expansion of the previously compressed lung; acute shortness of breath
Septa	Tissue bridges / chambering between separated spaces in a body cavity (e.g. abdomen)
Subcutaneous tissue	Tissue under the skin

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PleurX™/PeritX™ is a trademark and/or registered trademark of Becton Dickinson and Company, or one of its subsidiaries.
drainova® is a trademark and/or registered trademark of ewimed.
HMT Medizintechnik GmbH
Germany | ewimed GmbH | Dorfstrasse 24 | 72379 Hechingen | Tel.: +49 7471 73972-0 | info@ewimed.com
Sweden & Denmark | ewimed Sweden AB | Ekbacksvägen 28 | 168 69 Bromma | Tel.: +46 8 25 11 69 | info.se@ewimed.com