ewimed

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CONTENT

- 2 Explanation of symbols**
- 6 Product description**
- 16 The drainova® dressing set**
- 16 Warnings**
 - general
 - in relation to your catheter
- 17 Preparation**
- 18 Application of the dressing set**
- 20 Frequently asked questions**
- 20 Returns/refunds**
- 21 Glossary**

EN

Please read the entire instructions of use carefully before using the drainova® dressing set.

All explanations of terms (*) can be found in the glossary on page 21.

YOU HAVE QUESTIONS OR NEED HELP?

Contact us:

DE: +49 7471 73972-0
info@ewimed.com

AT: +43 2231 22500
info@ewimed.com

HU: +36 1 998 8295
info@ewimed.hu

EN: +49 7471 73972-0
info@ewimed.com

SE: +46 8 25 11 69
info.se@ewimed.com

CH: +41 52 577 02 55
info.ch@ewimed.com

DA: +45 31 43 50 60
info.dk@ewimed.com

For further detailed information please visit our website www.ewimed.com

THE DRAINOVA® DRESSING SET

The drainova® dressing set contains all necessary components for applying a dressing after carrying out a pleural or ascitic drainage in combination with a drainova®, PleurX™ or PeritX™ catheter. The drainova® Silicone cap is a protective cap used for sealing a drainova®/PleurX™/PeritX™ pleural or ascites catheter.

The drainova® dressing set is hereinafter referred to as the „dressing set“.

We recommend the following additional items for carrying out the drainage:

- Alcoholic disinfection spray
- Hand disinfectant
- Non-sterile disposable gloves

EN

WARNINGS

general

The dressing set is only suitable for use for the intended purpose.

Do not use this product if:

- the sterile packaging is damaged
- the product is damaged
- the expiry date has expired

The consumables are intended for single use only!

The dressing set is delivered in a sterile condition, which is ensured by appropriate quality controls. The dressing set must not be re-sterilised. The product is for single-use only. Re-use can lead to cross-contamination.

ewimed GmbH accepts no liability for products that have been re-used, re-sterilised or removed from damaged packaging.

Cleaning / disinfection

Should fluid leak out and come into contact with skin, clean the skin with soap and water.

Use suitable cleaning or disinfectant products for surfaces.

Do not use octenisept® or products containing iodine to clean / disinfect the catheter or safety valve.

The alcohol wipes are slightly flammable.

Keep the wipes away from naked flames.

WARNINGS

in relation to your catheter


Never cut through the catheter or cut the safety valve off.

Ensure that the catheter does not come into contact with any sharp objects.

If the functionality of the safety valve cannot be guaranteed or if the safety valve has been cut off by mistake (i.e. is no longer attached), proceed as follows:

- firmly pinch the catheter closed with your fingers.
- Hold the emergency clamp with the other hand and close the clamp with the inserted catheter until a click is heard.
- Immediately contact the implanting clinic.

EN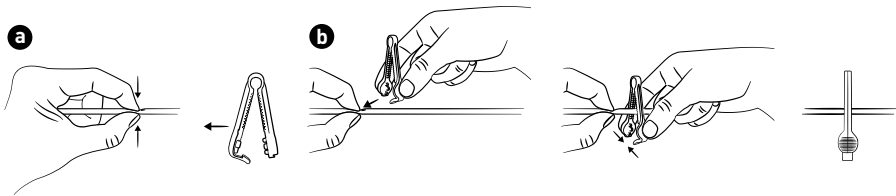

PREPARATION

- Prepare a clean, empty work space on a table or other work surface.

Please note: Place an open container nearby to dispose of used material.

- Disinfect your hands. We also recommend washing your hands thoroughly before disinfecting.
- If possible, put on non-sterile disposable gloves (not included in the dressing set).
- Remove the dressing covering the catheter. Carefully peel away a corner of the dressing and pull it slowly away from the skin, making

sure that the catheter is not accidentally pulled out.


Important: Inform your doctor should you notice swelling, signs of inflammation or an accumulation of fluid at the catheter exit site or incision site.

- Take off your gloves and disinfect your hands again.
- Open the sterile packaging of the dressing set by pulling the two sides of the bag apart at the top.


APPLICATION OF THE DRESSING SET

EN

- Place the dressing set with the flap facing up on the work surface. Carefully unfold the blue drape. Leave the contained items on the drape. The items in the dressing set are sterile and must not come into contact with non-sterile objects.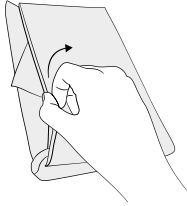

- Pick up the gloves at the opening and put them on as illustrated. Both gloves are suitable for either hand. Make sure that the gloves do not come into contact with non-sterile objects e.g. skin or clothing.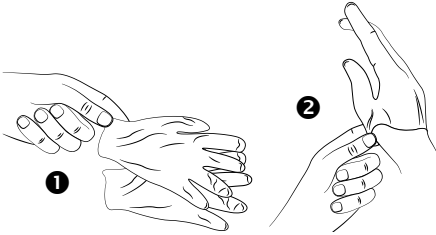

- 2.1 Tear open the alcohol wipe sachet but do not take the wipe out. Open the sachets so that these can be placed on the safety valve later on. Place the opened alcohol wipe sachets back on the edge of the blue drape.


- Remove the drainova® silicone cap, sterile or the PleurX™ / PeritX™ Valve Cap:

- **Using the drainova® silicone cap, sterile:** Firmly hold the safety valve of the catheter and carefully remove the drainova® silicone cap, sterile. Dispose of the cap.


- **Using the PleurX™ / PeritX™ Valve Cap:** Firmly hold the safety valve of the catheter, rotate the PleurX™ / PeritX™ Valve Cap

anticlockwise and carefully remove. Dispose of the cap.

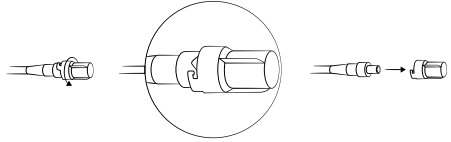
- Insert the safety valve into one of the open alcohol wipe sachets and clean the valve using gentle circular movements. Allow 30 seconds to take effect.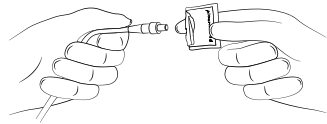

Warning: Do not use octenisept® or other disinfectants containing iodine.

- Open the packaging of the drainova® silicone cap, sterile and let the cap fall onto the blue drape.

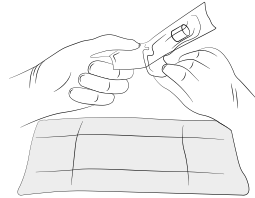
Please note: Drainage is to be carried out according to the separate ewimed, drainova® or PleurX™ Drainage set instructions of use.

- Attach the drainova® silicone cap, sterile: Securely attach the new sterile drainova® silicone cap onto the safety valve of the catheter. The cap will stay in place without the need for additional support.


- Clean the area around the catheter exit site with the skin disinfectant spray containing alcohol. If the wound has healed and the sutures have been removed, you can use an alcohol wipe. Please ensure that after

applying the disinfectant, the catheter exit site and the skin surrounding the catheter is dry. Only then should you apply the new dressing.

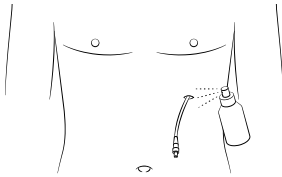
Disinfecting the pleural* catheter exit site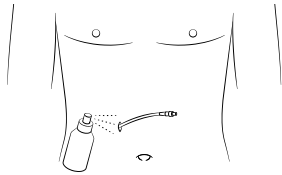


Disinfecting the ascites* catheter exit site


8. Place the foam pad with the slit facing upwards around the catheter.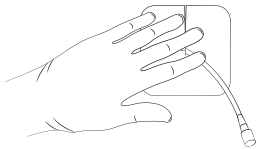

9. Loop up the catheter and place it on the foam pad.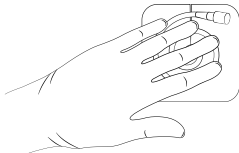

10. Cover the catheter with the gauze pad.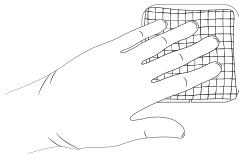


11. Take off both gloves.


12. The film dressing has a backing sheet on both sides.

- The printed backing sheet consists of a centre section and 2 side edge strips. The self-adhesive side of the film dressing is located underneath.
- The unprinted backing sheet consists of a centre section and an outer frame.


13. First remove the centre part of the unprinted backing sheet.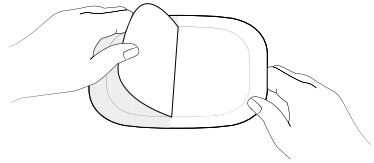


14. Then remove the centre part of the printed backing sheet to expose the adhesive side. You can do this by slightly bending the side edge strips.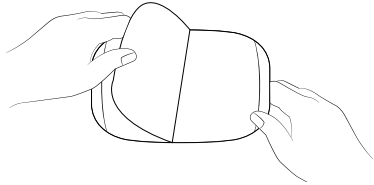

15. Hold the backing sheet on the edge strips. You now have a window on the unprinted side to help you position the film dressing. Stick the film dressing centrally over the catheter.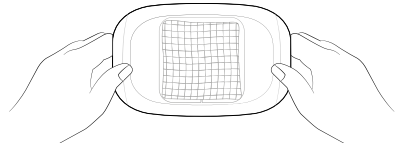

16. Next, remove the frame of the unprinted backing sheet and smooth the film dressing from the inside outwards (there is a perforated area on the edge to start peeling the frame off in a circle).

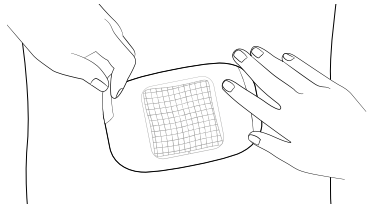EN

20 | Application of the dressing set

17. As a last step, remove the two edge strips by holding the film dressing (close to the edge strips) in place with one hand and then tearing the edge strips off in one piece with the other hand. The dressing is perforated at this point

and tears off together with the backing sheet. Finally smooth out the film dressing again.

Please note: Dispose of the used dressing according to the local regulations.

EN

FREQUENTLY ASKED QUESTIONS

When can the sutures (stiches) be removed?

The sutures can typically be removed after 7 – 10 days. The suture holding the catheter in place however, should not be removed before 30 days have elapsed.

Can I shower or have a bath with the catheter?

You can shower as long as the catheter is completely and correctly covered with the self-adhesive film dressing. If the gauze pads however do get wet during showering, remove the dressing, clean and dry the area and apply a new dressing. The catheter should not be completely immersed in water either in the bath or in the swimming pool. Avoid direct contact of the safety valve with bath/shower water.

Please note: We recommend showering before draining or applying a new dressing, so that the dressing can be promptly changed should it become wet.

How can I tell if I have an infection?

Immediately inform a doctor should you experience pain, skin redness (erythema*), if the skin is warm to the touch, swelling (oedema*), fever or accumulation of fluid as these can be signs of infection*.

Should you notice a change in colour of the effusion fluid, proceed as described.

Shortly after implantation redness and mild pain can be expected. These symptoms should not last or become worse.

What should I do if I accidentally pull out the catheter?

The catheter should have grown together with the subcutaneous skin tissue through the polyester cuff. If the catheter is accidentally pulled out and the cuff is visible, cover the exit site with a sterile dressing and immediately contact the clinic that carried out the implantation.

RETURNS / REFUNDS

Only originally packaged (unopened, non-damaged, non-labelled) boxes of drainage sets will be accepted for return and refund.

GLOSSARY

Ascites	Effusion or accumulation of fluid in the abdomen (stomach)
DEHP	Plasticizers in plastics
Edema	Swelling due to excessive fluid accumulation
Erythema	Skin redness
Infection	Reaction of the body's own defence system to pathogens such as bacteria, viruses etc.
Pleura	Thin, serous skin in the thoracic cavity (pleura)
Subcutaneous tissue	Tissue under the skin

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PleurX™/PeritX™ is a trademark and/or registered trademark of Becton Dickinson and Company, or one of its subsidiaries.
drainova® is a trademark and/or registered trademark of ewimed.
octenisept® is a trademark of Schülke & Mayr GmbH.
Albo-Healthcare GmbH
ewimed Switzerland AG
Fuhrmann GmbH