ewimed

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## CONTENT

- 2 Explanation of Symbols
- 7 Product Description
- 18 The drainova® Dressing set
  - general
  - in combination with the catheter
- 19 Preparation
- 20 Application of the dressing set
- 22 Frequently asked questions
- 23 Returns/refunds
- 23 Glossary

EN

**You should have already been instructed in the correct use of the components in this set either by ewimed training personnel, district nursing staff or a doctor. Please contact your doctor or district nurse if you are experiencing difficulties or have questions relating to effusion drainage.**

**Please read the entire instructions of use carefully before using the drainova® dressing set.**

All explanations of terms (\*) can be found in the glossary on page 23.

## YOU HAVE QUESTIONS OR NEED HELP?

Contact us:

<b>DE</b> : +49 7471 73972-0 info@ewimed.com	<b>SE</b> : +46 8 25 11 69 info.se@ewimed.com	<b>BE</b> : +49 7471 73972-0 info@ewimed.com
<b>EN</b> : +49 7471 73972-0 info@ewimed.com	<b>DA</b> : +45 31 43 50 60 info.dk@ewimed.com	<b>NL</b> : +49 7471 73972-0 info@ewimed.com
<b>CH</b> : +41 31 958 66 66 info.ch@ewimed.com	<b>NO</b> : +46 8 25 11 69 info.no@ewimed.com	<b>LU</b> : +49 7471 73972-0 info@ewimed.com
<b>AT</b> : +43 2231 22500 info@ewimed.com	<b>HU</b> : +36 1 998 8295 info@ewimed.hu	

For further detailed information please visit our website [www.ewimed.com](http://www.ewimed.com)

## THE DRAINOVA® DRESSING SET

The drainova® dressing set contains all necessary components for applying a dressing after carrying out a pleural or ascitic drainage in combination with a drainova®, drainova® Argenti©, PleurX™ or PeritX™ catheter.

The drainova® silicone cap, sterile is a protective cap used for sealing a drainova® / drainova® Argenti©, PleurX™ / PeritX™ catheter.

The drainova® dressing set is hereinafter referred to as the 'dressing set'.

### We recommend the following additional items for carrying out the drainage:

- Alcoholic disinfection spray
- Hand disinfectant
- Non-sterile disposable gloves

## WARNINGS

general

**The dressing set is only suitable for use for the intended purpose.**

### Do not use this product if:

- the sterile packaging is damaged
- The product is damaged
- the use-by date has elapsed

### The consumables are intended for single use only!

The dressing sets are delivered in a sterile condition, which is ensured by appropriate quality controls. The dressing sets can not be re-sterilised. The product is for single-use only. Re-use can lead to cross-contamination.

ewimed GmbH accepts no liability for products that have been re-used, re-sterilised or removed from damaged packaging.

### Cleaning / disinfection

If effusion is leaking out and comes in contact with skin, clean the skin with soap and water. For surfaces, use suitable cleaning products or surface disinfectant. Do not use octenisept® or disinfectants containing iodine for cleaning / disinfecting the catheter or safety valve.

### The alcohol swabs are slightly flammable.

Keep the swabs away from naked flames.

## WARNINGS


in combination with the catheter

### Never cut through the catheter or cut the safety valve off.

Avoid contact of the catheter with sharp objects.

If the functionality of the safety valve cannot be guaranteed or if the safety valve has been cut off by mistake (i. e. is no longer attached), proceed as follows:

- firmly pinch the catheter closed with your fingers.
- using your other hand, hold the emergency slide clamp open so that the catheter can be fed through the open end of the clamp.
- push the catheter completely through the narrow end of the clamp.
- Immediately inform your doctor.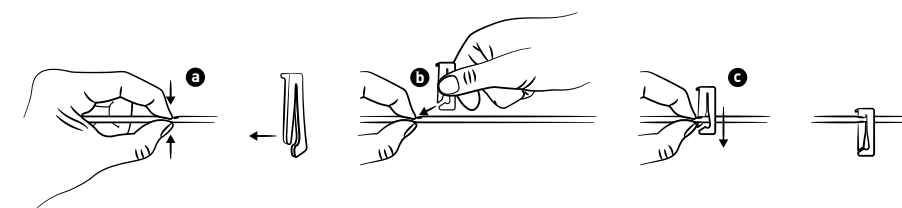

## PREPARATION

1. Prepare a clean, empty work space on a table or other work surface.

**Please note:** Place an open container nearby to dispose of used material.

2. Disinfect your hands. We also recommend washing your hands thoroughly before disinfecting.
3. Put on the non-sterile disposable gloves.
4. Remove the dressing covering the catheter. Carefully peel away a corner of the dressing and pull it slowly away from the skin,


making sure that the catheter is not accidentally pulled out.

**Important:** Inform your doctor should you notice swelling, signs of inflammation or an accumulation of fluid at the catheter exit site or incision site.

5. Take off your gloves and disinfect your hands again.
6. Open the sterile packaging of the dressing set by pulling the two sides of the bag apart at the top.

**APPLICATION OF THE DRESSING SET**

- Place the dressing set with the flap facing up on the work surface. Carefully unfold the surgical drape. Leave the contained items on the drape. The items in the dressing set are sterile and must not come into contact with non-sterile objects.

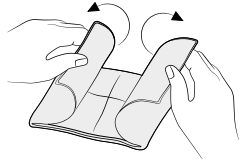
- Pick up the gloves at the opening and put them on as illustrated. Both gloves are suitable for either hand. Make sure that the gloves do not come into contact with non-sterile objects e.g. skin or clothing.


- 2.1. Tear open the alcohol swab sachet but do not take the wipe out. Open the sachets so that these can be placed on the safety valve later on. Place the opened alcohol swab sachets back on the edge of the drape.

- Remove the drainova® silicone cap, sterile or the PleurX™/ PeritX™ Valve Cap:


- When using the drainova® silicone Cap, sterile:** Firmly hold the safety valve of the catheter and carefully remove the drainova® silicone Cap, sterile. Dispose of the cap.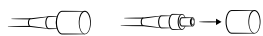

- When using the PleurX™/ PeritX™ Valve Cap:** Firmly hold the safety valve of the catheter, rotate the PleurX™/ PeritX™ Valve Cap anticlockwise and carefully remove. Dispose of the cap.

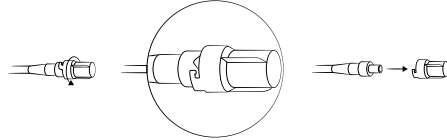
- Insert the safety valve into one of the open alcohol swab sachets and clean the valve using gentle circular movements. Allow 30 seconds to take effect.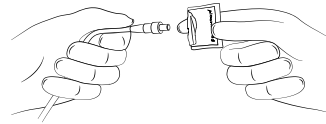

**Warning:** Do not use octenisept® or other disinfectants containing iodine.

- Open the packaging of the drainova® silicone cap, sterile and let the cap fall onto the drape.


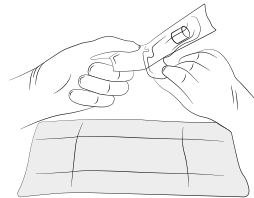
**Please note:** Drainage is to be carried out according to the separate ewimed, drainova® or PleurX™ Drainage set instructions of use.


- Attach the drainova® silicone cap, sterile: Securely attach the new sterile drainova® silicone cap onto the safety valve of the catheter. The cap will stay in place without the need for additional support.

- Clean the area around the catheter exit site with the skin disinfectant spray containing alcohol. If the wound has healed and the sutures have been removed, you can use an alcohol swab. Please ensure that after applying the disinfectant, the catheter exit site and the skin surrounding the catheter is dry. Only then should you apply the new dressing.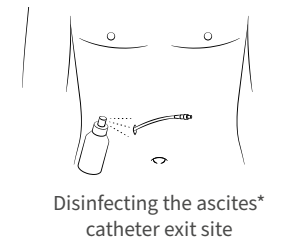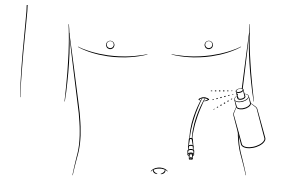


- Place the foam pad with the slit facing upwards around the catheter.

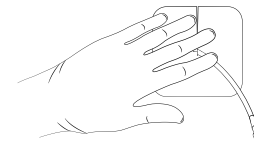
- Loop up the catheter and place it on the foam pad.


- Cover the catheter with the gauze pad.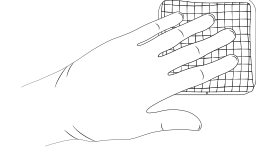


- Take off both gloves.
- The self-adhesive film dressing is comprised of three layers.
  - Non-printed protective film (two pieces)
  - Self-adhesive film dressing (transparent)
  - Printed protective film
- Expose the self-adhesive surface by peeling off the printed film.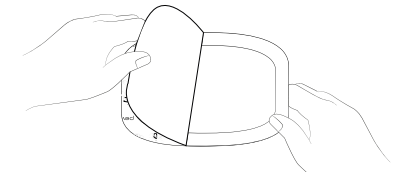


- Centre the self-adhesive film dressing over the gauze pads and attach to the surrounding skin.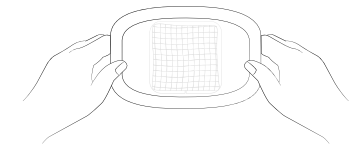

- Detach the middle part of the non-printed protective film from the film dressing.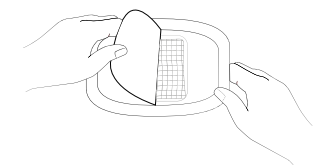

16. Carefully peel the edge of the non-printed protective film away from the film dressing and smooth out the entire self-adhesive film dressing starting from the centre to the edges.

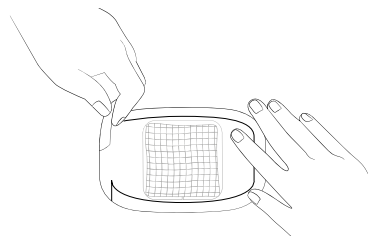**Please note:** Dispose of the used dressing according to the local regulations.

EN

## FREQUENTLY ASKED QUESTIONS

### When can the sutures (stiches) be removed?

The sutures can typically be removed after 7 – 10 days. The suture holding the catheter in place however, should not be removed before 30 days have elapsed.

### Can I shower or have a bath with the catheter?

You can shower as long as the catheter is completely and correctly covered with the self-adhesive film dressing. If the gauze pads however do get wet during showering, remove the dressing, clean and dry the area and apply a new dressing. The catheter should not be completely immersed in water either in the bath or in the swimming pool. Avoid direct contact of the safety valve with bath/shower water.

**Please note:** We recommend showering before draining or applying a new dressing, so that the dressing can be promptly changed should it become wet.

### How can I tell if I have an infection?

Immediately inform a doctor should you experience pain, skin redness (erythema\*), if the skin is warm to the touch, swelling (oedema\*), fever or accumulation of fluid as these can be signs of infection\*.

Should you notice a change in colour of the effusion fluid, proceed as described. Shortly after implantation redness and mild pain can be expected. These symptoms should not last or become worse.

### What should I do if I accidentally pull out the catheter?

The catheter should have grown together with the subcutaneous skin tissue through the polyester cuff. If the catheter is accidentally pulled out and the cuff is visible, cover the exit site with a sterile dressing and immediately contact the clinic that carried out the implantation.

## RETURNS / REFUNDS

Only originally packaged (unopened, non-damaged, non-labelled) boxes of drainage sets will be accepted for return and refund.

EN

## GLOSSARY

<b>Ascites</b>	Accumulation of fluid in the abdomen
<b>DEHP</b>	Softening agents in plastics
<b>Erythema</b>	Skin redness
<b>Infection</b>	Reaction of the body's own defence system to pathogens such as bacteria, viruses etc.
<b>Pleura</b>	Thin, serous skin in the thoracic cavity
<b>Subcutaneous tissue</b>	Tissue beneath the skin
<b>Oedema</b>	Swelling due to excessive accumulation of fluid
<b>Subcutaneous tissue</b>	Tissue beneath the skin

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PleurX™/PeritX™ is a trademark and/or registered trademark of Becton Dickinson and Company, or one of its subsidiaries.
drainova® is a trademark and/or registered trademark of ewimed GmbH.
octenisept® is a trademark of Schülke & Mayr GmbH.
Shaoxing Yibon Medical Co., Ltd.
OBELIS S.A.
OBELIS Swiss GmbH